

Recommended Cleaning Procedure
Rüsch Reusable Copper Stylet
Product # 008801000

After each use, the Rüsch reusable copper stylet (product # 008801000) should be cleaned and disinfected in the following manner.

1. Scrub stylet in detergent and warm water to remove all visible foreign material.
2. Rinse well in clean water.
3. The stylet is now ready for disinfection following institution's standard policy. The product may be processed with steam autoclave, Sterris™, Sterrad™, Ethylene Oxide, or cold soak disinfecting solutions. Regardless of which method of disinfection is chosen, care should be exercised to insure that all disinfecting solution/ material is removed from the stylet prior to the next use.