

Filiforms and Followers

Rusch filiforms and followers sold under the following part numbers can be processed for reuse:

- 3423xx Filiform with straight tip
- 3427xx Filiform with spiral tip
- 3433xx 20" Filiform with straight tip
- 3437xx 20" Filiform with spiral tip
- 3453xx Phillips Bougie Follower
- 3463xx Phillips Catheter Follower

Recommendations for Cleaning and Re-Processing:

- Clean thoroughly.
- Rinse abundantly and dry.

Cold Soak Method:

- Decontaminate using an appropriate solution, following the procedure recommended by the manufacturer of the decontaminant used.

Ethylene oxide sterilization:

Products are suitable for ETO gas sterilization. Equipment manufacturer's recommendations must be followed. Hospital protocol should be adhered to.

- ***A/ways make sure the product is in good condition and perfect working order before reusing it.***

To place an order, call our Customer Service Department or contact your Rüsçh representative at our toll free number.

1-800-514-7234

8:30 AM – 5:00 PM Eastern Time

Teleflex Medical

4024 Stirrup Creek – Suite 720, Durham, NC 27703

Telephone: 800-514-7234 Fax: 800-399-1028

www.rusch.com